

## **Hepro Standing Support chairs** For models S1 to S10



**Please read this operating instruction carefully!**

**Tested and in conformance with the following norms and guidelines:**

**EN 12182  
EN 1335-1, 2, 3  
EN ISO 14971**

**EN 1021 – 1  
EN 1021-2**



## Assembling

The chairs are delivered unassembled. Except S2 which is completely assembled.


### ASSEMBLY

- Put the wheelbase on the floor and the gas spring down in the center.
- Put the seat on the gas spring.
- If the model has a backrest or chest support, mount this:
  - Place the backrest gently on the post.
  - Press the clamp for the height regulation locking mechanism sideways.
  - Then press the backrest to it's lowest position.
  - The backrest is now correct assembled.
  - With the clamp pressed sideways the backrest can be regulated to the desired position .

Install the wheelbase on the floor with the wheels down. Then put the gas spring with the seat unit down in the center hole. To be sure it is stuck, press the seat unit firmly down.

### Regulate the brake handle on model S2


Loosen the two unbraco screws **O** to move the brake handle sideways. It can also be moved to the other side. The position is locked by turning the screws the other way. Use the 2,5 mm unbraco key delivered with the chair. (It is located on the right side of the wheelbase center). The chair is delivered with the brake handle totally in

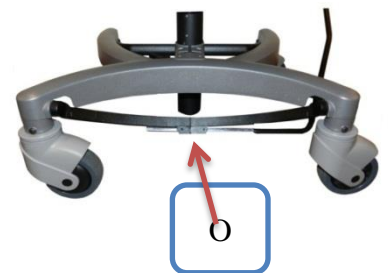

## Regulations:

### Models S3, S8 og S9

- Backrest height: - Press the locking clamp 1 sideways and the backrest can be regulated.
- Backrest angle and seat tilt: - Lift the lever 2 against the backrest. Then regulate the backrest angle and seat tilt to desired position. To lock this position, turn the lever forward and down again.
- Seat height: Lift the lever 3 for regulation. The position is locked when you let the lever down again.
- Chest angle and seat tilt: - As for backrest angle and seat tilt.
- Load brake wheels: Lock the wheels when you sit on the chair, and unlock them when rising up again.

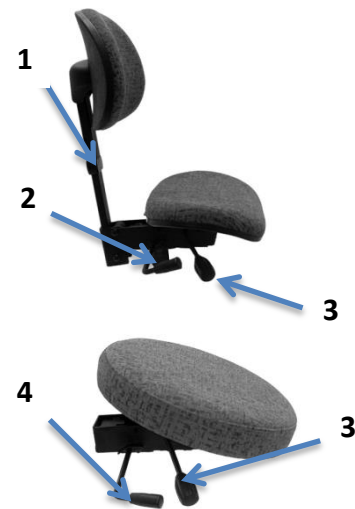

### Models S1, S2, S6 and S10

- Backrest height: - Press the locking clamp 1 sideways and the backrest can be regulated.
- Backrest angle: Lift the lever (2) against the backrest. Then regulate the angle. To lock this position, turn the lever forward and down again.
- Seat tilt: Lift the lever (3) against the backrest. Then regulate the seat angle. To lock this position, turn the lever forward and down again.
- Seat height: Lift the lever 4 for regulation. The position is locked when you let the lever down again.
- Central brake on model S2: Press the brake handle forward to lock the wheels for both rolling and rotation. The brake is released when the brake handle is pushed backwards.



# Standing Support Chair User Manual



## Cleaning

The fabric can be washed in water up to 60°C.

All parts of the frame and chassis can be cleaned with a damp cloth and mild soap.

## Warranty

When used in accordance with the Operating Instruction HEPRO provides a 2-year warranty against defects in material and workmanship.

Please address warranty complains or other inquiries about the product to your supplier or importer. Always state the product`s serial number as a reference.

## Safety Provisions

- The chair is an aid and shall not be used as a «toy» for children!
- Exceeding approved user weight (120 kg) may cause damage to both user and product!
- All 5 load brake wheels lock when sitting on the chair and unlock when rising again. It is important to be familiar with this function before start using the chair.
- On models with foot support: Don`t stand on it, the chair will probably overturn!
- Although the fabric is flame retardant it must not be exposed to naked flame, cigarette embers or the like!